

color

Kit Koleksyon Tès COVID-19 ki pa kontwole

Enstriksyon ki Sou Sit la

Anvan ou kòmanse:

- Ou dwe depoze echantiyon ou nan depatman koleksyon tès ki deziyen an menm jou koleksyon an.
- Asire w genyen yon koleksyon jodi a. Tcheke avèk kolektè tès COVID-19 ki pral aksepte echantiyon yo jodi a.
- Pa manke dènye ramasaj la. Tcheke avèk kolektè tès COVID-19 sou dènye lè koleksyon an.
- Asire w lave men ou!

Kit ou an gen:



Swab



Collection tube



Barcode card



Biohazard bag

1

Aktive kit ou sou entènèt nan color.com/covid/activate.

Ou bezwen kòd inik ou, ou pral jwenn nan tib koleksyon ou.



- ! **Atansyon: Echantiyon ou pa pral teste si kit la pa aktive.**


2

Anvan w kolekte echantiyon ou, lave men ou byen pandan 20 segonn epi byen seche yo.

3

Retire bouchon an nan tib koleksyon an.

Antenha a tampa perto.

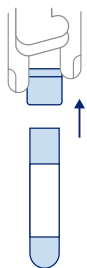


4

Louvri pake aplikatè a.

Kòmans nan fen manch lan nan pake prelevman, louvri fèy papye a. Retire aplikatè a nan anbalaj nan manch lan.

Nòt: Pa manyen pwent an koton an ak men w oswa pa depoze l sou nenpòt sifas.

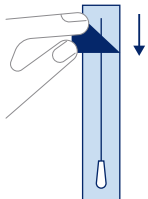


5

Vire aplikatè a nan lye twou nen w, 3 fwa.

Mete li nan yon sèl twou nen jiskaske pwent an koton an pa vizib ankò sou 1 pous. Vire li nan yon sèk alantou kwen an, andedan twou nen w omwen 3 fwa.

Nòt: Pa fòse prelevman an anpil.

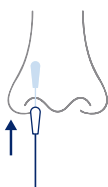


6

Vire pwent koton an nan lòt twou nen w, 3 fwa.

Repete premye etap anvan an nan dezyèm twou nen, sèvi avèk lòt pwent aplikatè a.

Nòt: Ou pral sèvi avèk menm aplikatè a pou toude twou nen yo.

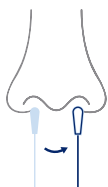

7

Mete aplikatè a nan tib la.

Pwent koton aplikatè a ki te rantre nan nen w ta dwe ale nan tib la tousuit.




8

Mete bouchon tib koleksyon an epi fèmen l.

Pouse bouchon tib koleksyon an pou fèmen l! Asire w tèt la fèmen sere.

Nòt: Apre tib koleksyon an fin fèmen, lave men w byen pandan 20 segonn epi byen seche l.

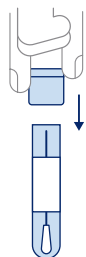

9

Mete tib koleksyon an nan sachè biohazard.

Mare sachè a.



10

Pote sak biohazard a nan depatman koleksyon tè s ou te chwazi a.

Koleksyon tè s COVID-19 ou a pral montre w ki kote pou w depoze echantiyon ou.

Nòt: Si w rate dènye tan ranmasaj la jodi a, mande kolektè tè s COVID-19 ou a yon nouvo kit.



Lis kontwòl anvan ou remèt kit ou an:

- Èske w te aktive kit ou?
- Èske w te kolekte yon echantiyon pou toude twou nen yo?

Vizite color.com/sample-return pou aprann plis.

Kesyon?

Vizite support.color.com oswa kontakte nou nan support@color.com oswa (844) 352-6567.

Anvan ou kòmanse:

- Ou dwe depoze echantiyon ou nan depatman koleksyon tès ki deziyen an menm jou koleksyon an.
- Asire w genyen yon koleksyon jodi a. Tcheke avèk kolektè tès COVID-19 ki pral aksepte echantiyon yo jodi a.
- Pa manke dènye ramasaj la. Tcheke avèk kolektè tès COVID-19 sou dènye lè koleksyon an.
- Asire w lave men ou!

Kit ou an gen:



Swab



Collection tube



Barcode card

Biohazard
bag

1

Aktive kit ou sou entènèt nan color.com/covid/activate.

Ou bezwen kòd inik ou, ou pral jwenn nan tib koleksyon ou.



! Atansyon: Echantiyon ou pa pral teste si kit la pa aktive.


2

Anvan w kolekte echantiyon ou, lave men ou byen pandan 20 segonn epi byen seche yo.

3

Retire bouchon an nan tib koleksyon an.

Antenha a tampa perto.

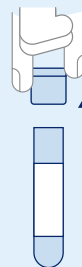



4

Louvri pake aplikatè a.

Kòmanse nan fen manch lan nan pake prelèyman, louvri fèy papye a. Retire aplikatè a nan anbalaj nan manch lan.

Nòt: Pa manyen pwent an koton an ak men w oswa pa depoze l sou nenpòt sifas.

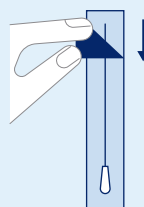

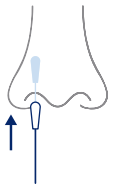


5

Vire aplikatè a nan 1ye twou nen w, 3 fwa.

Mete li nan yon sèl twou nen jiskaske pwent an koton an pa vizib ankò sou 1 pous. Vire li nan yon sèk alantou kwen an, andedan twou nen w omwen 3 fwa.

Nòt: Pa fòse prelevman an anpil.

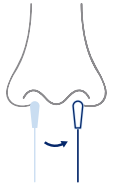

6

Vire pwent koton an nan lòt twou nen w, 3 fwa.

Repete premye etap anvan an nan dezyèm twou nen, sèvi avèk lòt pwent aplikatè a.


Nòt: Ou pral sèvi avèk menm aplikatè a pou toude twou nen yo.



7

Mete aplikatè a nan tib la.

Pwent koton aplikatè a ki te rantre nan nen w ta dwe ale nan tib la tousuit.

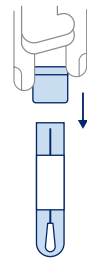

8

Mete bouchon tib koleksyon an epi fèmen l.

Pouse bouchon tib koleksyon an pou fèmen l! Asire w tèt la fèmen sere.

Nòt: Apre tib koleksyon an fin fèmen, lave men w byen pandan 20 segonn epi byen seche l.



9

Mete tib koleksyon an nan sachè biohazard.

Mare sachè a.

10



Pote sak biohazard a nan depatman koleksyon tè s ou te chwazi a.

Koleksyon tè s COVID-19 ou a pral montre w ki kote pou w depoze echantiyon ou.

Nòt: Si w rate dènye tan ranmasaj la jodi a, mande kolektè tè s COVID-19 ou a yon nouvo kit.

Lis kontwòl anvan ou remèt kit ou an:

- Èske w te aktive kit ou?
- Èske w te kolekte yon echantiyon pou toude twou nen yo?

Vizite color.com/sample-return pou aprann plis.

Kesyon?

Vizite support.color.com oswa kontakte nou nan support@color.com oswa (844) 352-6567.

Sèlman pou Famasi itilize, Sèlman pou IVD itilize, Pou itilize sèlman si w gen otorizasyon pou ijans. Kit koleksyon pou lakay sa a pa gen apwobasyon oswa fe vè FDA. Kit koleksyon pou lakay sa a te gen otorizasyon FDA dapre yon EUA.

Kit koleksyon pou lakay sa a te gen otorizasyon pou se sèlman lakay w pran epi kenbe echantiyon nan nen kòm yon èd pou detekte asid nikleyik nan SARS-CoV-2, pa pou lòt viris oswa mikwòb.

Kit koleksyon adomisil sa a gen otorizasyon pou sèlman dire deklarasyon egzistans ki jistifye otorizasyon itilizasyon an ijans pou dyagnostik in vitro pou deteksyon ak dyagnostik COVID-19 dapre atik 564(b)(1) lwa, 21 U.S.C. § 360bbb-3(b)(1), amwenske otorizasyon an anile oswa tan l fini pi bonè.